


Made in CZECH REPUBLIC



CONTENT OF PACKAGE


- 1 pc ergonomic headband
- 1 pc reinforcing chin
- 1 pc plexiglas with protective foil (must be removed before assembly)
- 1 pc rubber for fastening to the head
- 1 pc instructions how to assemble, maintain and use protective shield

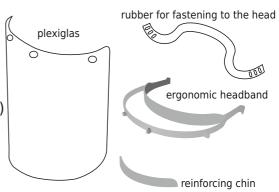


HOW TO ASSEMBLE

- * We recommend disinfecting all parts before assembling
- 1. remove the protective foil from the plexiglass and then clean it with disinfectant
- 2. attach the plexiglass to pre-prepared holes
- 3. gently slide the "chin" from below onto the edge of the plexiglas
- 4. put on the rubber and adjust your length depending on the holes



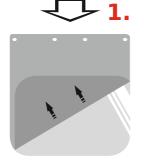

WWW.PROTECTOR19.com

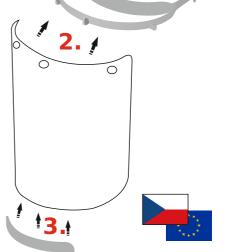

Thank you for your purchasing protective shield PROTECTOR19



Made in CZECH REPUBLIC



USE

Protective shield PROTECTOR19 serves as face protection and protects your eyes and mouth from direct contact with other particles.

Serves as a partial barrier from viruses and bacterias, prevents contact from splashing water during hobby activities.

Does not exclude any harmful fumes, can be easily desinfected, shield for repeated use.

HOW TO TAKE A PROTECTIVE SHIELD

It is recommended that all protective shield components be disinfected and cleaned before use. While we are trying to ensure as sterile conditions as possible for packing the shield, it is important to avoid any risk of contamination. We do not recommend using aggressive chemistry for disinfection!

! IMPORTANT INFORMATION!

Face protection PROTECTOR19 is **not for use** for industrial work, such as work with an angle grinder, chainsaw, welding technique, etc. In this case, use is at its own risk.





PROTECTIVE SHIELD **PROTECTOR19** - www.PROTECTOR19.com